Instructions for use

Patient target group
All persons for whom individual trays are required for treatment.

Intended user
Dentist

Composition Qualitative data:
Oligomeric acrylate vinyl ester resin, 2,2-dimethoxy-1,2-diphenylethan-1-one or
2,2-bis(2-methyl-2-phenyl)acetonaphenone, (+)-2,3-dioxo-1,7,7-trimethyl-bicyclo[2.2.1]-heptan-2-one, diphenyl-2,4,6-trimethylbenzylphosphonoxide,
2-hydroxy-2-methyl-1-phenyl-propone-1, 2-ethylhexyl-4-(dimethylamino)benzoate,
2-ethylhexyl-4-(dimethylamino)benzoate, dyes (Pigment Red 146 = C12485 / Pigment Blue 29 = CI 77007 / D&C Red 30 = CI 73360 / Pigment Blue CI 74160 / Pigment Violet 19 = C.I. 73900), flavorings (Optimant).

Quantitative data:
In 100 g there are 19 g acrylates/methacrylates and 80 g glass.

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Non usage:
Palatray XL non è light-curing tray material in the form of prefabricated, baseplates to make light, custom impression trays and bite-blocks, as well as for other indications.

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2-hydroxy-2-methyl-1-phenyl-propone-1, 2-ethylhexyl-4-(dimethylamino)benzoate,

2-ethylhexyl-4-(dimethylamino)benzoate, dyes (Pigment Red 146 = C12485 / Pigment Blue 29 = CI 77007 / D&C Red 30 = CI 73360 / Pigment Blue CI 74160 / Pigment Violet 19 = C.I. 73900), flavorings (Optimant).

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- Bite-block

- Base para setting up dentures

- Key for supporting pin registration

- Brackets for try-in framework

- Splicing outer coping or telescopic crowns

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Horarios:

Working by daylight 20 minutes

Polymerisation in an ultraviolet or Halogen-light-curing unit, depending on the unit in use:

The top of the tray, on the model 3-5 minutes

The underside of the tray, off the model 3-5 minutes

Trimming:

- Trim the custom tray with tungsten carbide burs and smooth it with sandpaper.

- Hygiene impressions taken with Palatray XL customs trays in a disinfectant solution based on glutaraldehyde.

- Remove the baseplate from the tray and close it.

- Open the tray and remove the baseplate to cut it to shape using, e.g. a scalpel.

- Use the remaining material to form a handle and attach it to the tray.

- Paint the tray with resin.

- Remove the tray from the model and polymerize once again from the underside.

- If the tray has been cured without a vacuum, use alcohol to remove the smear layer.

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