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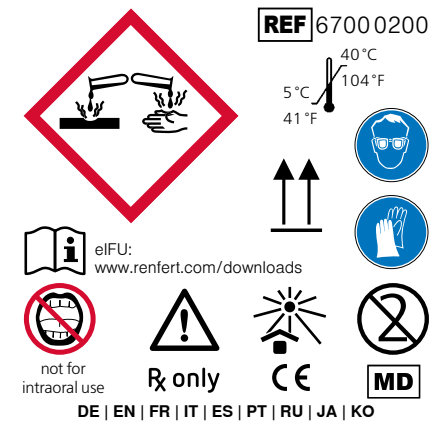
ENGLISH

Danger. Contains: 2-aminoethanol. May be corrosive to metals. Causes severe skin burns and eye damage. Wear protective gloves and eye/face protection. IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water or shower. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER.

Product description: Ready-to-use special cleaner (alkaline) for the subsequent removal of stubborn discolorations such as tar, coffee, tea stains from removable dental restorations.

Indication: Special cleaner for dissolving / removing stubborn discolorations such as tar, coffee, tea stains from removable dental restorations. Only to be used by trained dental personnel.

Contraindication: Application in the mouth or on the body is not permitted. The use of the cleaning agent on restorations belonging to patients with a known allergy to individual constituents is contraindicated.

Safety instructions: The owner and user is responsible for proper disinfection of the cleaning beakers and lids as well as for compliance with national hygiene regulations.

Special hazard warnings: Please read the cleaning agent safety data sheet before use. Download under www.renferf.com May be corrosive to metals. Causes severe skin burns and eye damage.

Wear protective gloves and eye/face protection. IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water or shower. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do, continue rinsing. Immediately call a POISON CENTER/doctor.

The liquids must not come into contact with the following surfaces: PVC, linoleum, natural and artificial stone, marble, stainless steel, wood. In case of contamination and spillage, clean immediately with a copious amount of water in order to avoid material damage.

In case of complaints or events in connection with the medical device, please inform Renferf immediately. Renferf will investigate the matter and report serious incidents to the authorities in the member states concerned.

Disclaimer: This product has been developed and is intended exclusively for professional use by trained dental professionals. It must be handled in accordance with the information provided in this instruction document.

Dosage and application: Before use, it is essential to read the comprehensive Instructions for Use of the SYMPRO high-performance cleaning device. Download under www.renferf.com help:ex discolor f is intended for single-use only, in order to prevent cross contamination.

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Unique Device Identifier

R only Professional use only

Observe the Instructions for Use: eIFU: www.renferf.com/downloads

EC conformity

Medical device

Manufacturer specifications

Not intended for reuse

Batch designation

Attention: observe accompanying document eIFU: www.renferf.com/downloads

Shelf life

Storage temperature

Protect against sunlight

Use protective eyewear

Wear protective gloves

Do not use orally

Corrosive - Danger

Store upright

UDI

Unique Device Identifier

R only Professional use only

Observe the Instructions for Use: eIFU: www.renferf.com/downloads

EC conformity

Medical device

Manufacturer specifications

Not intended for reuse

Batch designation

Attention: observe accompanying document eIFU: www.renferf.com/downloads

Shelf life

Storage temperature

Protect against sunlight

Use protective eyewear

Wear protective gloves

Do not use orally

Corrosive - Danger

Store upright

UDI

Unique Device Identifier

R only Professional use only

Observe the Instructions for Use: eIFU: www.renferf.com/downloads

EC conformity

Medical device

Manufacturer specifications

Not intended for reuse

Batch designation

Attention: observe accompanying document eIFU: www.renferf.com/downloads

Shelf life

Storage temperature

Protect against sunlight

Use protective eyewear

Wear protective gloves

Do not use orally

Corrosive - Danger

Store upright

UDI

Unique Device Identifier

R only Professional use only

Observe the Instructions for Use: eIFU: www.renferf.com/downloads

EC conformity

Medical device

Manufacturer specifications

Not intended for reuse

Batch designation

Attention: observe accompanying document eIFU: www.renferf.com/downloads

Shelf life

Storage temperature

Protect against sunlight

Use protective eyewear

Wear protective gloves

Do not use orally

Corrosive - Danger

Store upright

UDI

Unique Device Identifier

R only Professional use only

Observe the Instructions for Use: eIFU: www.renferf.com/downloads

EC conformity

Medical device

Manufacturer specifications

Not intended for reuse

Batch designation

Attention: observe accompanying document eIFU: www.renferf.com/downloads

Shelf life

Storage temperature

Protect against sunlight

Use protective eyewear

Wear protective gloves

Do not use orally

Corrosive - Danger

Store upright

UDI

Unique Device Identifier

R only Professional use only

Observe the Instructions for Use: eIFU: www.renferf.com/downloads

EC conformity

Medical device

Manufacturer specifications

Not intended for reuse

Batch designation

Attention: observe accompanying document eIFU: www.renferf.com/downloads

Shelf life

Storage temperature

Protect against sunlight

Use protective eyewear

Wear protective gloves

Do not use orally

Corrosive - Danger

Store upright

UDI

Unique Device Identifier

R only Professional use only

Observe the Instructions for Use: eIFU: www.renferf.com/downloads

EC conformity

Medical device

Manufacturer specifications

Not intended for reuse

Batch designation

Attention: observe accompanying document eIFU: www.renferf.com/downloads

Shelf life

Storage temperature

Protect against sunlight

Use protective eyewear

Wear protective gloves

Do not use orally

Corrosive - Danger

Store upright

UDI

Unique Device Identifier

R only Professional use only

Observe the Instructions for Use: eIFU: www.renferf.com/downloads

EC conformity

Medical device

Manufacturer specifications

Not intended for reuse

Batch designation

Attention: observe accompanying document eIFU: www.renferf.com/downloads

Shelf life

Storage temperature

Protect against sunlight

Use protective eyewear

Wear protective gloves

Do not use orally

Corrosive - Danger

Store upright

UDI

Unique Device Identifier

R only Professional use only

Observe the Instructions for Use: eIFU: www.renferf.com/downloads

EC conformity

Medical device

Manufacturer specifications

Not intended for reuse

Batch designation

Attention: observe accompanying document eIFU: www.renferf.com/downloads

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