dima® Print Stone beige Smart

- DE Gebrauchsanweisung
- GB Instructions for use
- FR Mode d'emploi
- ES Instrucciones de uso
- IT Istruzioni per l'uso
- PT Instruções de uso

Kulzer GmbH
Leipziger Straße 2
63450 Hanau (Germany)

+49 800 4372522

Made in Germany

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Instructions for use

1. Intended purpose

Photopolymer for the fabrication of dental models. For use by dental health care professionals only.

1.1 Indications

For 3D fabrication of dental models

1.2 Contraindications

Not for use in the mouth. In case of known or suspected allergy to components of the product, its use is contraindicated.

1.3 Potential side effects

Hypersensitivities to the product or its components cannot be excluded in individual cases.

2. Product features

2.1 Package sizes/Supply Forms

1000g bottle

2.2 Accessories

The material can be printed using printers with a wavelength of 405 or 385 nm (currently released devices – see table below):
The cleaning takes place in the cleaning device, e.g. cara Print CleanUp or ultrasonic bath. Post-curing of the material must be carried out with a suitable exposure device.

3D printer / Manufacturer		Post-curing unit / Manufacturer	
• cara Print Cube • cara Print 4.0 <i>pro</i>	Kulzer	• HiLite power 3D • cara Print LEDcure	Kulzer

Always use the latest firmware version of your 3D printer.

Layer thickness: 100µm / 150 µm

2.3 Composition

Contents: (Meth-)acrylate containing monomers, photoinitiators, pigments

3. Product use

3.1 Processing properties

Materials	dima Print Stone beige Smart
Homogenization	Shake for five minutes
Cleaning solution	Isopropanol (pure)
Pre-cleaning	3 min.
Post-cleaning (fresh rinse)	2 min.
Post-curing	• HiLite power 3D 2 x 5 min (1x top, 1x bottom) • cara Print LEDcure in accordance with the exposure program

3.2 Application subitems

When filling the vat or when refilling the liquid into the bottle, it must not be mixed with other photopolymers, as otherwise negative changes in the product properties (colour, mechanical and chemical properties) may occur.
a. Shake vigorously in the original container for five minutes before use.
Inadequate shaking may result in color variations and printing errors.

dima

Print Stone beige Smart

b. Print the objects according to the instruction manual supplied with the 3D printer.

Before starting printing, wait until any air bubbles have disappeared.

c. Thoroughly remove remaining liquids using compressed air.

d. Pre-clean the printed objects for three minutes in Isopropanol using the appropriate supplies (see Item 2.2).
Note: Thoroughly rinse the inside of the structures with the spray bottle (Isopropanol).

e. Repeat step d using pure Isopropanol for two minutes.
Note: The objects should not remain in the Isopropanol for more than five minutes, otherwise they will be damaged.
Note: Thoroughly rinse the inside of the structures again with the spray bottle (Isopropanol). After cleaning, make sure the printed objects are dry and free from solvent residues.

f. Place the printed object for final polymerization in the photo polymerization unit according to the table above.
Note: The final properties depend on the adherence to the recommended post-curing process.

4. General information

Please, carefully observe the following safety instructions and all processing notes in other sections. This product is intended to be used only in accordance with this Instructions for Use. Any use of this product inconsistent with these instructions is at the discretion and sole responsibility of the dental health care professional.

4.1 Warnings and safety precautions

Causes skin irritation. Causes serious eye damage.
May cause an allergic skin reaction. Harmful to aquatic life with long lasting effects. Wash thoroughly after handling. Avoid release to the environment. Wear protective gloves / eye protection / face protection. If on skin: Wash with plenty of water. If in eyes: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If skin irritation or rash occurs: Get medical advice/attention. Do not breathe dust/fume/gas/mist/vapours/spray. Wear protective equipment. Keep unprotected persons away. Use breathing protection against the effects of fumes/dust/aerosol. Avoid contact with eyes and skin. Do not breathe vapor / mist / gas. Ensure adequate ventilation. May cause damage to organs through prolonged or repeated exposure. Immediately call a POISON CENTER/doctor.

4.2 Storage conditions

Do not use the material after the expiration date. Do not store above room temperature of 15° (59°F) - 30 ° (86°F). Keep away from sunlight. Store in a dry place.

4.3 Disposal Information

Recommendation: Dispose of in accordance with official regulations.
Do not dispose of contents or partially emptied packaging in the household waste or allow them to enter the sewage system European Waste Catalogue: 180106 Chemicals consisting of or containing hazardous substances.

5. Complaint notice

Please quote batch number and the article number in all correspondence about the product. Article number and batch number: See note on packaging.

The safety instructions in this document do not replace consultation of the Material Safety Data Sheet (MSDS). You can obtain this from your supplier or in the download area at:
www.kulzer.com/cara-print

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