OPAKER

DILUENTE PER OPAKER

OPACIZZANTE PER METALLI

OPACIFIER FOR METALS

OPACIFICANTE PARA METAIS

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OPACIFICANTE PARA METALES

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(I) Istruzioni d'uso

(GB) Instructions for Use

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(D) Gebrauchsanweisung

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DILUENTE PER OPAKER

(GB) DIRECTIONS:

Acrylic preparation for masking the metallic part of dental prostheses before applying the aesthetic coating in acrylic resin.

CONTRAINDICATIONS:

- Any other use not provided for in these instructions.
- In case of proven allergy to components of Opaker.

INSTRUCTIONS:

Shake vigorously before use. The metal must be sandblasted, perfectly cleaned and degreased in the part to be opacified. Apply a thin and uniform layer of Opaker using a brush:

- apply a very thin first coat (add *Diluente per Opaker* to obtain the desired consistency of Opaker);
- leave to dry;
- apply another layer until completely covered.

Leave to dry at room temperature for at least 10 minutes. To facilitate evaporation of the solvent, expose the metal covered with Opaker to a moderate source of heat for several minutes (do not place on an open flame).

IMPORTANT: before proceeding to apply the resin for bridges and crowns, eliminate any edges with a sharp blade and ensure that the solvent in the Opaker is completely evaporated. Use the thinner to facilitate cleaning of the instruments.

Store at a maximum temperature of 30°C.

The product is designed for professional use in the dental sector. Follow the instructions for use. The manufacturer shall accept no responsibility for damages arising from different or inadequate use. The user must personally check the suitability of the product for the intended use, particularly if these uses are not reported in the instructions for use. Do not use after expiration. Warning: Any serious accident that may occur with the device must be reported by the user to the manufacturer and the competent authority of the Member State where the user and/or patient are established.

Highly flammable liquid and vapour. Causes skin irritation. May cause an allergic skin reaction. May cause respiratory irritation. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Avoid breathing mist/vapours/spray. Wear protective gloves/protective clothing/eye protection/face protection. IF ON SKIN: Wash with plenty of soap and water. Store in a closed container. Dispose of contents/container in accordance with local/regional/national/international regulations.

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Ruthinium® - Dental Manufacturing S.p.A.